



Get Your Life Back



TIES[®] Transcutaneous Implant Evacuation System

TIES[®] is an alternative to a traditional ostomy and has been developed to improve the quality of life for ileostomy patients. The system is intended to minimize the need for a permanent bag and the associated problems and inconveniences this may entail.

No Permanent Bag – Greater Freedom and Control

The TIES[®] system works in a similar way to the human body in that you decide when you want to empty, either into a bag or directly into the toilet. If preferred, a conventional ostomy bag can also be used, for example during the night.

TIES[®] is the result of several years of research combined with the latest manufacturing technology. The system is developed in Norway and Sweden, and is manufactured in accordance with current regulations to ensure patient safety.

The TIES[®] implant consists of an external titanium ring with an internal porous surface structure which, with the help of 3D technology, is built layer by layer into a solid part using a pure material without impurities. TIES[®] has a unique structure with optimized surfaces to promote healing. These factors ensure reliable tissue in-growth and minimize the risk of infection for you as a patient.

Only for clinical trial
NOT FOR COMMERCIAL USE



Co-funded by the Horizon 2020
programme of the European Union

Is TIES[®] the solution for you?

Get your life back.



TIES[®] Transcutaneous Implant Evacuation System

The TIES[®] solution can give you greater control over your Ileostomy. Patients where TIES[®] can fit usually suffer from Crohn's disease, ulcerative colitis or other bowel disorders. TIES[®] can be used for both creating a new stoma and for converting an existing stoma. Ask your doctor if TIES[®] is suitable for you. You can also email: info@ostomycure.com or visit our website to find a list of participating clinics and sign up for more information.



This is how TIES[®] works

First, the surgeon implants a TIES[®] titanium ring within the stoma. The titanium ring acts as a holder for the TIES[®] lid. Following an initial healing period after surgery, you can apply the lid to the titanium ring by yourself. The lid has a hatch that you can open and close to empty the stoma. When the hatch is closed there is no leakage, no unwanted sounds and no "accidents". When you feel it is time to empty, you simply attach a specially designed bag and then open the hatch. The contents are then transferred to the bag, or directly into the toilet, without any soiling or leaks.



 **ostomycure**

Participating clinics in UK,
Sweden, Poland and Austria.
Visit our website for more details:
www.ostomycure.com



No: 2133145



info@ostomycure.com



Gaustadalléen 21
0349 OSLO, Norway